



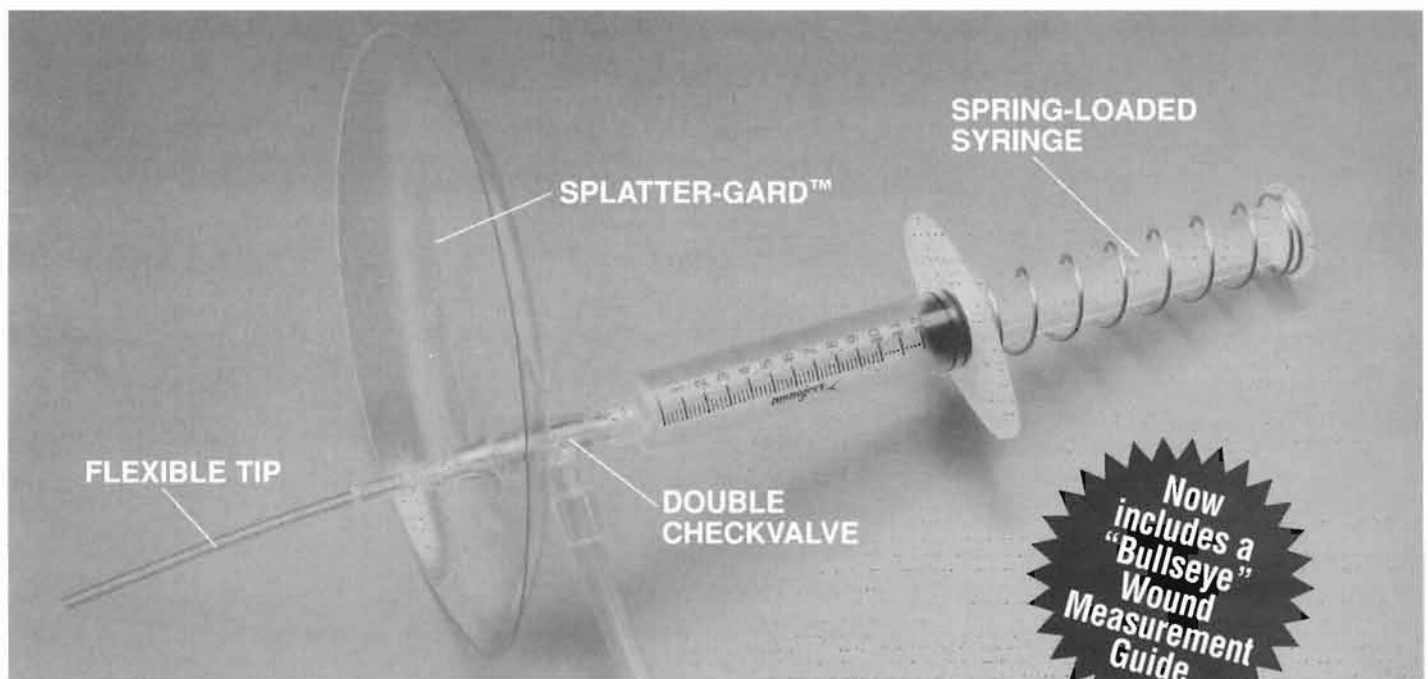

# IRRIJET® DS

Delivers optimal pressure-ulcer irrigation through a continuous sterile pathway.

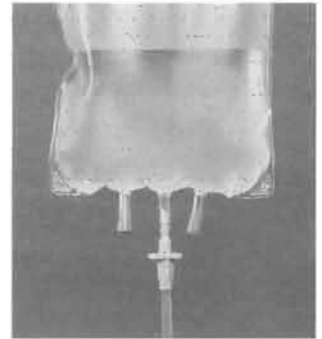

- Soft, flexible 2<sup>3</sup>/<sub>4</sub>" tip eliminates needle-stick for user, minimizes trauma to patient
- Splatter-Gard™ prevents contamination from body secretions—now with new concave design
- Closed system provides sterile saline delivery up to 5 days
- Irrijet DS is listed in AHCPH Guidelines for Pressure Ulcer Treatment as providing recommended range of irrigation pressure

- Disposable, easy-to-use Irrijet DS is ideal for home care applications


The Irrijet DS is a unique, spring-loaded, self-refilling system that quickly and easily irrigates pressure ulcers. Simple, one hand, thumb-controlled pressure provides an appropriately forceful 7.6 psi stream directly to the desired area (or a gentler stream if needed). The soft flexible tip has an inside diameter equivalent to an 18-19 gauge needle and eliminates the need for an over-the-needle plastic sheath.

# Ordering Information


## IRRIJET DS PRODUCT NO. 55-6504

### ■ One irrigating assembly


A spring-loaded syringe is connected through a double checkvalve to a flexible plastic tip, through a side arm and 6 foot length of tubing to a spike for entry into a bag of irrigating solution. Syringe capacity, 12cc. Double checkvalve prevents retrograde flow in the plastic tip. Alternate movement of the syringe plunger delivers irrigating solution as needed.

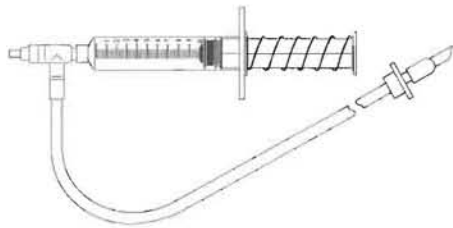

IRRIGATING ASSEMBLY

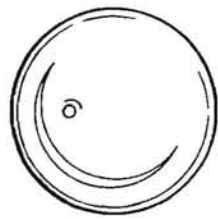

SPLATTER-GARD

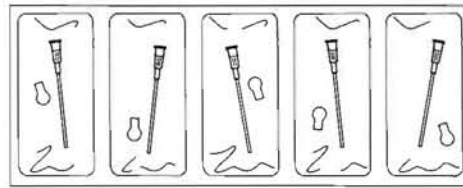

JET TIPS & CAPS



WOUND  
MEASUREMENT  
GUIDE

### ■ One SPLATTER-GARD™

### ■ Five 2<sup>3</sup>/<sub>4</sub>" jet tips

### ■ Five caps

### ■ One 10cm "Bullseye" Wound Measurement Guide

### PACKAGING

Sterile, 5 sets per box

4 boxes (20 sets) per case


## SPLATTER-GARD, TIP AND CAP SET PRODUCT NO. 55-6100

### ■ Individually wrapped packages, each containing a Splatter-Gard, jet tip and cap.

### PACKAGING

Sterile, 10 sets per box

4 boxes (40 sets) per case

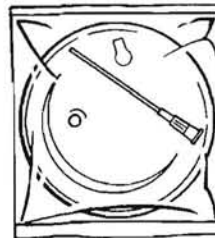

### ACKRAD LABORATORIES, INC.

70 Jackson Drive

Cranford, New Jersey 07016

Tel: 908-276-6390 ■ Fax: 800-293-9476

<http://www.ackrad.com>

Distributed By:

**IMA**  
Independent Medical Associates

Independent Medical Associates

11733 66th Street North  
Suite 113  
Largo, FL 33773

Phone: 888-548-4462  
Fax: 888-548-1462  
E-Mail: [Info@I-MA.com](mailto:Info@I-MA.com)