



Catheter Securement

Application of Grip-Lok Hydrocolloid for Umbilical Catheter Securement

Grip-Lok 2300NUCH Neonatal Umbilical Catheter Hydrocolloid stabilization device is designed to secure catheters made from silicone and other plastic compounds. The 2300NUCH holds the catheter upright without contacting the stump. *Neonatal Skin Care Practice Guidelines* state the preference for hydrocolloid adhesives on neonatal patients.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin according to the standard hospital protocol for dressing application.

1



Remove the Grip-Lok from the bottom release liner by lifting the fabric and top liner at the same time.

2




Place the Grip-Lok on the skin circling the umbilicus and remove the top liner to expose the adhesive areas.

3



Fold up to 90 degrees the semicircle of fabric and place catheter in the center of the exposed adhesive.

4

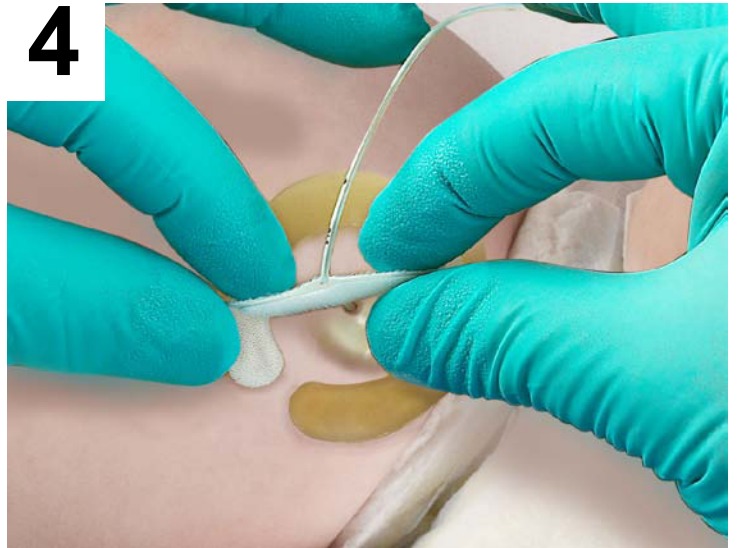

Fold up the opposite fabric to meet the semicircle and secure the catheter between the two layers of adhesive.



Upright catheter position is maintained by the Grip-Lok Neonatal Umbilical Catheter Hydrocolloid stabilization device

IMA
Independent Medical Associates

11733 66th St N Suite 113
Largo, FL 334773
888.548.4462
FAX: 888.548.1462
email: info@I-MA.com
website: www.I-MA.com
MM00035 Rev.1 2011-3