

## Application of Grip-Lok Large for Foley Catheter Securement

Grip-Lok Large (3300L) offers firm securement of Foley catheters except pure silicone materials and will fit a wide range of catheter sizes. Grip-Lok is applied in the same simple manner as a bandage and can be easily placed and adjusted with gloved hands.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin is according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

**1**



Pull the paper backing from one side of the Grip-Lok. Then pull the paper backing from the other, to secure the Grip-Lok in desired position on the skin.

**2**




Open the top Velcro® over-strap and remove the paper liners to expose adhesive strips.

**3**



Place the catheter centered on the bottom adhesive area.

**4**

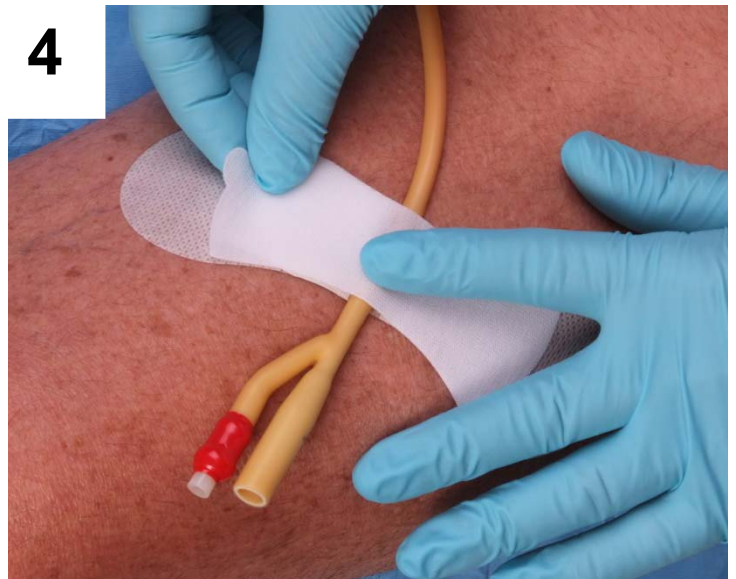

Secure top strap over the catheter so that it is placed between adhesive areas and top strap is closed.