Nasal Gastric Securement Device

REF

Description

2100NGH Nasal Gastric Hydrocolloid Universal Securement Device

STERILE R

LV07903 Rev 3

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

EC REP

Blue Box Medical Limited
Unit 29, New Forest Enterprise Centre,
Rushington Business Park
Chapel Lane
Totton, Southampton, SO40 9LA
United Kingdom



Phone: +023 8066 9000
Fax: +023 8066 9002
www.blueboxmedical.co.uk

Grip-Lok is a registered trademark of Zefon International.

INSTRUCTIONS

English

- Select the area for the placement of the Grip-Lok. Note: The feeding tube should be already inserted in the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Remove the bottom liner by holding fabric section and pulling-off liner. Do not remove the top liner at this time.
- Place the Grip-Lok on the site by continuing to hold by the fabric and press onto the skin with gentle pressure. The top liner should now be removed.
- Position the feeding tube in the center of the top exposed adhesive strip between the fabric and the white hook section.
- Secure the feeding tube by folding over the fabric section until it meets with the white hook portion and apply gentle pressure to the back of the entire fabric area.

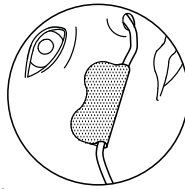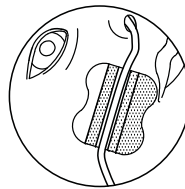Note: to remove or adjust secured tube, hold hydrocolloid adhesive down to skin while opening top fabric section.

Note: the Grip-Lok 2100NGH may be cut in half prior to removal of liners to reduce the size.

Note: replace securement device if soiled or saturated in fluid.

Note: use of a sterile water swab will ease removal of securement device from the skin.

This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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