## Universal Securement Device

h

Description

3300MEPI Medium Epidural Catheter Securement

C

0086

D

K

LV05016 Rev 1

Independent Medical Associates

M

Zefon International, Inc.  
5350 SW 1st Lane  
Ocala, FL 34474 USA  
  
Phone: +1 352 854 8080  
Fax: +1 352 854 7480  
www.zefon.com

P

Blue Box Medical Limited  
Unit 29, New Forest Enterprise Centre,  
Rushington Business Park  
Chapel Lane  
Totton, Southampton, SO40 9LA  
  
Phone: +023 8066 9000  
Fax: +023 8066 9002  
www.blueboxmedical.co.uk

INSTRUCTIONS



English

- Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open the top flap and hold the flap open.
- Position the Grip-Lok with the catheter in the groove at edge of the foam section.
- Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Remove the interior liner and place the catheter over the foam and onto the center of the exposed adhesive surface.
- Secure the top flap over the catheter and apply gentle pressure to the center fabric area over the catheter adhesive.

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: replace securement device if soiled or saturated in fluid.

Note: use of an alcohol swab will ease removal of securement device from the skin.

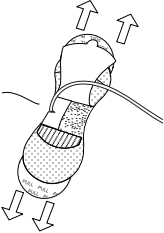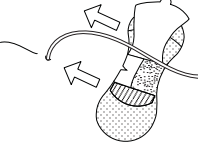This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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