## Securement Device for the BD Arterial Cannula with FloSwitch™

### h Description

3301MCS-BD Securement Device for the BD Arterial Cannula with FloSwitch™

C

0086

D

BD and FloSwitch trademarks are property of Becton, Dickinson and Company. Grip-Lok is a registered trademark of Zefon International.

K

LV05013 Rev.1

# IMA

Independent Medical Associates

M

Zefon International, Inc.  
5350 SW 1st Lane  
Ocala, FL 34474 USA

Phone: +1 352 854 8080  
Fax: +1 352 854 7480  
www.zefon.com

P

Blue Box Medical Limited  
Unit 29, New Forest Enterprise Centre,  
Rushington Business Park  
Chapel Lane  
Totton, Southampton, SO40 9LA

Phone: +023 8066 9000  
Fax: +023 8066 9002  
www.blueboxmedical.co.uk

## INSTRUCTIONS




English

- Select** the area for the placement of the Grip-Lok.  
Note: The arterial cannula should be already inserted in the patient.
- Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open** the top flap, remove the interior liner and hold the flap open.
- Slide** the Grip-Lok under the arterial cannula and adhere in the exposed adhesive cut-out space provided.
- Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Secure** the top flap over the arterial cannula with the FloSwitch portion coming through the center cut-out.

**Direction:** for added skin protection, skin prep may be used prior to application of securement device.

**Note:** replace securement device if soiled or saturated in fluid.

**Note:** use of an alcohol swab will ease removal of securement device from the skin.

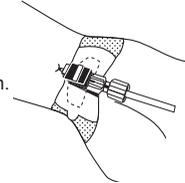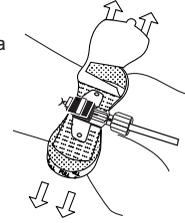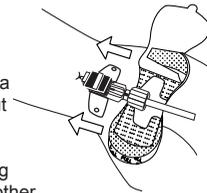This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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