Universal Securement Device

REF	Description
3200S	Small Universal Securement
3300M	Medium Universal Securement
3400L	Large Universal Securement

STERILE R

LV05006 Rev 3

IMA

Independent Medical Associates

www.I-MA.com
info@I-MA.com
p: 888.548.4462
f: 888-548-1462

INSTRUCTIONS

English

1. Select the correct size Grip-Lok for your application.

REF	Size	Line/Tubing Diameter
3200S	Small	1.5 to 4.5 mm
3300M	Medium	3.0 to 8.0 mm
3400L	Large	6.0 to 13.0 mm

2. Select the area for the placement of the Grip-Lok. Note: The line or tube should be already inserted in the patient.
3. Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
4. Open the top flap, remove the interior liner and hold the flap open.
5. Slide the Grip-Lok under so that the line or tube is in the center of the exposed adhesive.
6. Secure the top flap over the line or tube and apply gentle pressure to the center fabric area over the adhesive.
7. Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

Direction: for added skin protection, use skin prep pad prior to application of stabilization device.

Note: replace securement device if soiled or saturated in fluid.

Note: use of an alcohol swab will ease removal of securement device from the skin

This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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