


Easy to Use


Site Preparation

Remove hair, clean and dry skin. For best adhesion, use skin prep.

Apply

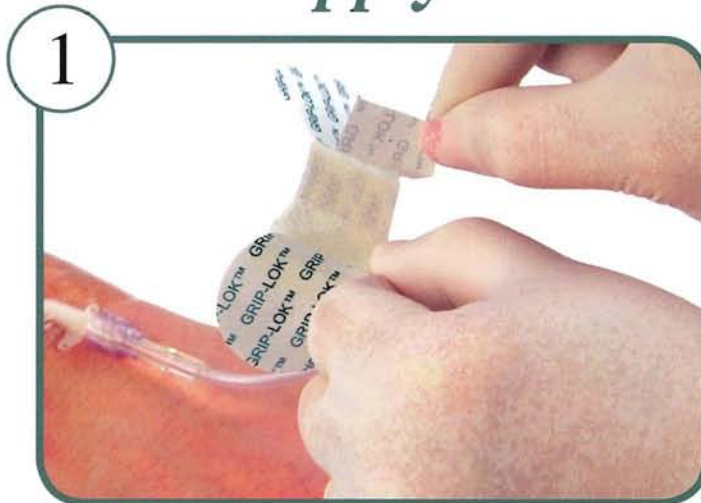

Peel the paper backing from the GRIP-LOK and secure in desired position on the skin.

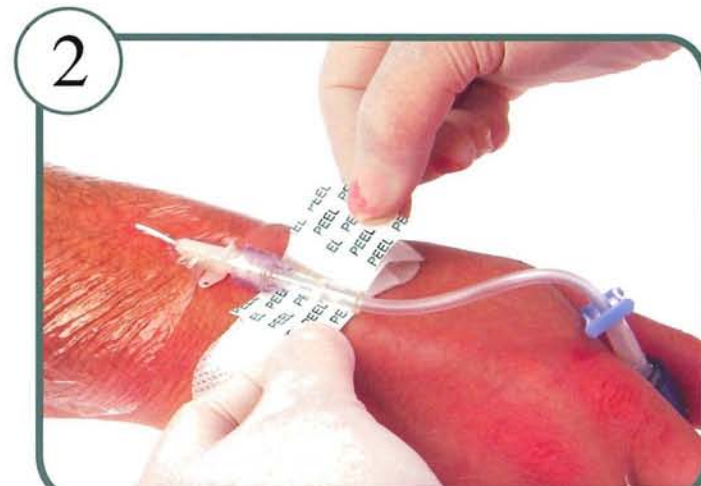

GRIP-LOK

Universal Securement

*For securing all types of lines,
tubes and catheters.*

GRIP-LOK offers many advantages over tape. It is designed to hold more securely than tape and reduce unscheduled restarts. Every line pull or IV restart that is avoided results in better patient care.



Lift the top flap and remove the paper backing to expose the adhesive strips.

3



Place tubing/hub on bottom adhesive strip.

4




Secure top flap over the tubing/hub so that tubing/hub is placed between adhesive strips and top flap is secured.

Remove

No special solution is required - gently remove GRIP-LOK, taking care not to pull on IV tubing.

GRIP-LOK is applied in the same simple manner as a bandage and can be easily placed and adjusted by gloved hands.

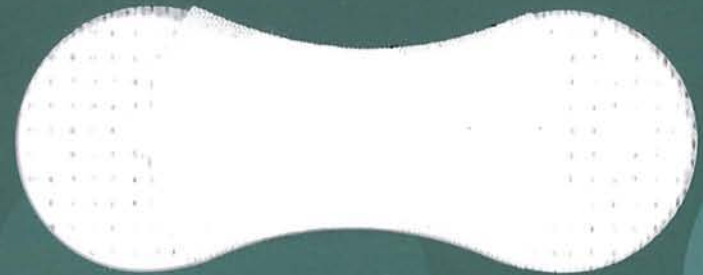

IMA

Independent Medical Associates

11733 66th St N #113 Largo, FL 33773

P: 888-548-4462 ; F: 888-548-1462

info@I-MA.com ; www.I-MA.com

For safe and proper use of this device, please refer to the complete instructions for use.

MM00011 Rev.0