## Securement Device for PICC/CVC

| h              | Description  |
|----------------|--|
| 33xxMCS Series | Medium Catheter Securement (MCS) for PICC/CVC Applications |

C

0086

D

Grip-Lok is a trademark of Zefon International, Inc.

K

LV05015 Rev.1

### INSTRUCTIONS




English

- Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
- Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open** the top flap, remove the interior liner and hold the flap open.
- Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
- Secure** the top flap over the catheter.
- Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

**Direction:** for added skin protection, skin prep may be used prior to application of securement device.

**Note:** replace securement device if soiled or saturated in fluid.

**Note:** use of an alcohol swab will ease removal of securement device from the skin.

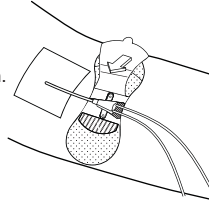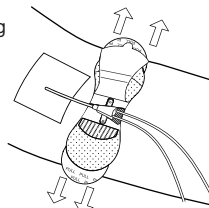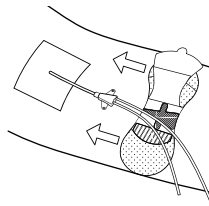This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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